

Installation Instructions | MediWatch Slim Metal Magnetic Straps

Please Read Before Installation:

We strongly recommend taking the device and the relevant accessories to a watch shop, as a successful installation demands a certain level of dexterity.

Step One: Removing Original Watch Straps (if applicable)

1. Turn over the MediWatch Slim and find the **quick-release lever** on one of the straps.
2. While pressing the quick-release lever inwards (towards the middle of the watch), gently pull the band away from the Watch to release it.
3. Repeat on the other side.



Step Two: Installing the Metal Magnetic Straps

1. Lay the strap flat.
2. Remove the loose attachment bar from the strap by sliding it off the magnet end.
3. Insert one of the spring bars into the loose attachment bar (as shown in the image).
4. With the watch face down, identify the side with the SOS button. The loose attachment bar is to be positioned above this.
5. Whilst holding the quick-release lever, place the other end of the spring bar next to its corresponding hole on the watch.
6. Carefully slide the bar into position and then release the quick-release lever. You may need to wiggle it around a little until it clicks into place.
7. Insert the second spring bar into the non-magnet end of the strap.
8. On the other side of the watch, repeat steps 5 and 6.



If you have further questions, please do not hesitate to call us at (08) 6336 9448.