

Installation Instructions | MediWatch Slim Silicone Magnetic Straps

Step One: Removing Original Watch Straps (if applicable)

1. Turn over the MediWatch and find the quick-release lever on one of the straps.
2. While pressing the **quick release lever** inwards (towards the middle of the watch), gently pull the band away from the watch.
3. Repeat on the other side.



Step Two: Installing the Silicone Straps

1. With the watch face down, identify the side with the SOS button.
2. With the SOS button on the left, attach the strap end with a loop **above** the SOS button.
3. Make sure the quick-release lever is facing upwards. Place the non-lever end of the spring bar into its corresponding hole on the watch (as shown in the image).
4. While pressing the quick-release lever inwards, slide the strap into place.
5. On the other side of the watch, repeat steps 3 and 4 with the longer strap.



If you have further questions, please do not hesitate to call us on (08) 6336 9448.