

## Installation Instructions | MediWatch Slim Velcro Straps

### Step One: Removing Original Watch Straps (if applicable)

1. Turn over the MediWatch Slim and find the quick-release lever on one of the straps.
2. While pressing the **quick release lever** inwards (towards the middle of the watch), gently pull the band away from the watch.
3. Repeat on the other side.



### Step Two: Installing the Velcro Straps

**Note:** the quick-release pin can sometimes slot into the recess of the attachments and may seem stuck. A pin or something similar can be wedged underneath it to free the pin if required.

1. With the watch face down, identify the side with the SOS button.
2. With the SOS button on the left, attach the strap end without the Velcro tabs above the SOS button.
3. Pick up the MediWatch Slim and attach the 'floating' end to the other end of the MediWatch Slim.
4. Place the wrist through the strap.
5. Adjust the strap length as needed. It's best practice to have at least 2 of the 5 Velcro tabs connected while wearing the strap.
6. All done!



If you have further questions, please do not hesitate to call us on (08) 6336 9448.